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Test With Confidence™

IDEXX

Avian Pneumovirus Antibody Test Kit

For veterinary use only.

Name and Intended Use

IDEXX APV is IDEXX's enzyme immunoassay for the detection of antibody to avian pneumovirus (APV) in chicken and turkey serum.

General Information

Avian Pneumovirus (APV) is a pathogen of chickens and turkeys that can produce respiratory signs and reproductive failures as a primary pathogen, or as part of a multiagent syndrome known as Swollen Head Syndrome (SHS). Virus isolation of APV is a difficult task, so diagnosis through serology is one of the most important tools for disease control. This test is intended for quantitative monitoring chicken and turkey flocks that have been vaccinated with live and/or inactivated products containing A, B or C antigens, as well as naive (SPF) flocks for the detection of infected birds with any A, B or C serotype of APV.

Descriptions and Principles

This assay is designed to measure the relative level of antibody to APV in chicken and turkey serum. Viral antigen is coated on 96-well plates. Upon incubation of the test sample in the coated well, antibody specific to APV forms a complex with the coated viral antigens. After washing away unbound material from the wells, a conjugate is added that binds to any attached chicken and turkey antibody in the wells. Unbound conjugate is washed away and enzyme substrate is added. Subsequent color development is directly related to the amount of antibody to APV present in the test sample.

Reagents		Volume
1	APV Antigen coated plate	5
2	Positive Control	1 x 1.9 mL
3	Negative Control	1 x 1.9 mL
4	Conjugate	1 x 50 mL
5	Sample Diluent	1 x 235 mL
A	TMB Substrate	1 x 60 mL
B	Stop Solution	1 x 60 mL

Note: See table at the end of the insert for a description of symbols used on the insert and labels of this kit.

Storage

Store the reagents at 2–8°C. Reagents are stable until expiration date, provided they have been stored properly.

Materials Required but Not Provided

- Precision micropipettes or multi-dispensing micropipettes
- Disposable pipette tips
- 96-well microplate reader (equipped with 650 nm filter)
- Microplate washer (manual, semi-automatic or automatic system)
- Use only distilled or deionized water for preparation of the reagents used in the test
- Microplate covers (lid, aluminium foil or adhesive)
- Vortex or equivalent
- Tubes for diluting samples.

Precautions and Warnings

- Handle all biological material as potentially infectious.
- Wear protective gloves / protective clothing / eye or face protection when handling samples and reagents.
- Refer to the product Material Safety Data Sheet for additional information.
- See the end of this insert for reagent hazard and precaution warnings.

Laboratory Practices

- Optimal results will be obtained by strict adherence to this protocol. Careful pipetting, timing, and washing throughout this procedure are necessary to maintain precision and accuracy. Use a separate pipette tip for each sample and control.
- Do not expose TMB solution to strong light or any oxidizing agents. Handle TMB solution with clean glass or plastic ware.
- All wastes should be properly decontaminated prior to disposal. Dispose of contents in accordance with local, regional, and national regulations.
- Care should be taken to prevent contamination of kit components. Do not pour unused reagents back into containers.
- Do not use kit past expiration date and do not intermix components from kits with different lot numbers.

Preparation of Samples

Dilute test samples five hundred fold (1:500) with sample diluent prior to being assayed (e.g., by diluting 1 μ L of sample with 500 μ L of Sample Diluent). **NOTE: DO NOT DILUTE CONTROLS.** Be sure to change tips for each sample. Samples must be thoroughly mixed prior to dispensing into the coated plate.

Test Procedure

All reagents must be allowed to come to 18–26°C before use. Mix reagents by gentle inverting or swirling.

1 Obtain antigen-coated plate(s) and record the sample position.

2 Dispense 100 μ l of UNDILUTED Negative Control (NC) into duplicate wells.

3 Dispense 100 μ l of UNDILUTED Positive Control (PC) into duplicate wells.

4 Dispense 100 μ l of DILUTED sample into applicable wells.

5 Incubate for 30 minutes (\pm 2 min.) at 18–26°C.

6 Remove the solution and wash each well with approximately 350 μ L of distilled or deionized water 3–5 times. Avoid plate drying between plate washings and prior to the addition of the next reagent. Tap each plate onto absorbent material after the final wash to remove any residual wash fluid.

7 Dispense 100 μ l of Conjugate into each well.

8 Incubate for 30 minutes (\pm 2 min.) at 18–26°C.

9 Repeat step 6.

10 Dispense 100 μ l of TMB Substrate into each well.

11 Incubate for 15 minutes (\pm 1 min.) at 18–26°C.

12 Dispense 100 μ l of Stop Solution into each well.

13 Measure and record absorbance values at 650 nm, A(650).

14 Calculations:

Controls

$$NC\bar{x} = \frac{NC1 A(650) + NC2 A(650)}{2}$$

$$PC\bar{x} = \frac{PC1 A(650) + PC2 A(650)}{2}$$

Validity criteria

$$PC\bar{x} - NC\bar{x} > 0.075$$

$$NC\bar{x} \leq 0.150$$

For invalid assays, technique may be suspect and the assay should be repeated following a thorough review of the package insert.

Samples

$$S/P = \frac{\text{Sample Mean} - NC\bar{x}}{PC\bar{x} - NC\bar{x}}$$

$$\text{Log}_{10} \text{ Titer} = 1.09 (\log_{10} S/P) + 3.36^*$$

*Relates S/P at a 1:500 dilution to an endpoint titer.

The presence or absence of antibody to APV is determined by relating the A(650) value of the sample to the Positive Control mean. The Positive Control is standardized and represents significant antibody levels to APV in chicken and turkey serum. The relative level of antibody in the sample is determined by calculating the sample to positive (S/P) ratio. Endpoint titers are calculated using the equation described above.

15 Interpretation:

Negative

Positive

$$S/P \leq 0.20$$

$$S/P > 0.20$$

A positive result (titer greater than 396) indicates vaccination or other exposure to APV. Each laboratory should establish its own criterion for immunity with respect to antibody titer based on correlation of IDEXX APV to current laboratory test methodologies and on historical antibody responses to specific vaccines and vaccination protocols. The immune status of a flock is best assessed by monitoring and recording antibody titers in representative samples as a function of time. The resulting flock profiles allow an assessment of the distribution of antibody titers and an analysis of changes in titer over time.

Note: IDEXX has instrument and software systems available which calculate results and provide data summaries.

For technical assistance:

IDEXX USA Tel: +1 800 548 9997 or +1 207 556 4895

IDEXX Europe Tel: +800 727 43399

Contact your IDEXX area manager or distributor or visit our website: idexx.com/contactlpd

IDEXX and Test With Confidence are trademarks or registered trademarks of IDEXX Laboratories, Inc. or its affiliates in the United States and/or other countries

© 2016 IDEXX Laboratories, Inc. All rights reserved.

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WARNING / ATTENTION / ATENCIÓN / ATENÇÃO / ACHTUNG

H315 / H319 / P280 / P332+P313 / P337+P313

TMB Substrate – Causes skin irritation. Causes serious eye irritation. Wear protective gloves/eye protection/face protection. If skin irritation occurs: Get medical advice/attention. If eye irritation persists: Get medical advice/attention.

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H316 / P332+P313 / EUH208

Conjugate – Causes mild skin irritation. If skin irritation occurs: Get medical advice/attention. Contains Proclin. May produce an allergic reaction.

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IDEXX Laboratories, Inc.
One IDEXX Drive
Westbrook, Maine 04092
USA

Manufacturer
IDEXX Switzerland AG
Stationsstrasse 12
CH-3097 Liebefeld-Bern
Switzerland

EU-Representative
IDEXX Europe B.V.
P.O. Box 1334
2130 EK Hoofddorp
The Netherlands

idexx.com

IDEXX