

BUSTER Tubular Bandage

This superb bandage has endless possibilities of usage. The diversity in sizes and ability to cut anywhere without fraying, makes it very versatile for both small animal and equine wounds.

- High elasticity and excellent material memory allow good fixation of bandages
- Very soft and comfortable for the patient
- Very easy to apply
- Does not unravel
- Very good compliance from owner



Unique Features

Very comfortable

BUSTER Tubular Bandage is made of a high quality knit, which is very soft and elastic and conforms to the shape of the animal.

Very elastic

The elasticity makes the bandages stay firmly in place without any further fixations.

Easy to apply

Simply cut the BUSTER Tubular Bandage in the length needed and adjust to the shape of the bandaged area. It is possible to use dressings on even difficult-to-bandage places like tails.

Skin friendly

BUSTER Tubular Bandage is porous and permits air-flow, which gives a comfortable environment on the skin.

Reliable and stable fixation over a long period of time

The BUSTER Tubular Bandage has excellent material memory. If the bandage loosens, simply remove and stretch the bandage length-wise to gain initial shape.

Does not unravel

The special knitting technic of BUSTER Tubular Bandage makes it possible to cut anywhere and anyhow in the bandage without fraying.

Good compliance from the patient

The BUSTER Tubular Bandage is very light, does not constrict and is not sticky.

The animal cannot tear the bandage with its paws, as the bandage does not fray.



Very elastic



Very elastic



Does not unravel

Recommended use for BUSTER Tubular Bandages

- Controlled fixations of bandages
- Fixating dressings on difficult places like ears, tails, after abdominal surgery, cattle eye patch and many others
- Regular fixations of e.g. IV-catheters

Instructions for use

Application

Evaluate the size of BUSTER Tubular bandage taking into consideration the circumference of area.

The length of the bandage will get slightly shorter, when the bandage expands.

Be careful if you have to cut in the bandage already applied on the animal.

Can be reused.

Use BUSTER Tubular Bandage as secondary/tertiary bandage on top of primary/secondary dressing.

Primary dressing suggestions

- KRUUSE Hydro Gel
- KRUUSE Manuka Honey

Secondary dressing suggestion

- KRUUSE Foam Dressing

Can also be used in combination with

- KRUUSE Vet-Plast
- KRUUSE Vet-Flex
- KRUUSE Soft-Flex bandages



Removal

Unroll bandage or cut gently with a bandage scissor, be careful not to cut into the skin.

Be aware of compression during application and wearing. If the bandage is applied with too small circumference of the bandage or with too many layers over a small part of the body.

Apply evenly over a larger area

Cat. No	Description
160480	BUSTER Tubular Stretch Bandage, white, 1.25 cm x 6.5 m
160481	BUSTER Tubular Stretch Bandage, white, 2.5 cm x 6.5 m
160482	BUSTER Tubular Stretch Bandage, white, 3.8 cm x 6.5 m
160483	BUSTER Tubular Stretch Bandage, white, 5 cm x 6.5 m
160484	BUSTER Tubular Stretch Bandage, white, 7.5 cm x 6.5 m
160485	BUSTER Tubular Stretch Bandage, white, 8.8 cm x 6.5 m



For veterinary use only