


SMI-SPON

ABSORBABLE GELATIN SPONGE

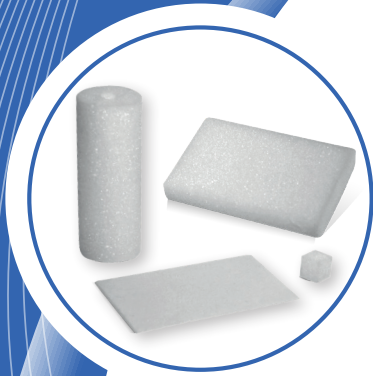

What is SMI-SPON

SMI-SPON is a surgical haemostatic sponge, manufactured from highly purified first grade gelatin material for use in various surgical procedures.

It is a quick and effective haemostatic which stops bleeding fast. It reduces intraoperative bleeding and surgery time.

Why using SMI-SPON

SMI-SPON can be used in various surgeries for haemostasis. When applied in vivo and used in appropriate amounts, it is completely absorbed within 3-4 weeks. SMI-SPON is biocompatible, non-toxic and non-pyrogenic.

SMI-SPON is gamma sterilized and individually packed in ready to use blister packaging.

Where to use

- Dental surgery
- Plastic surgery
- Vascular surgery
- Orthopaedic surgery
- ENT surgery
- Neuro surgery
- Abdominal surgery
- Anorectal surgery
- Genito Urinary surgery
- Gynaecological surgery

How to use SMI-SPON

- Cut a piece to the desired size
- Apply SMI-SPON either in a dry state or saturated with physiological salt solution
- If used dry, a piece of SMI-SPON is applied to the bleeding place and held in place with moderate pressure until hemostasis results
- If used wet, first immerse in the solution and then withdraw, squeeze between gloved fingers to expel air bubbles, and the replace in saline until needed

Standard 80 x 50 x 10 mm



*For quality
&
safety*

smi
www.sutures.be