

VETIGEL®

VETIGEL® is a hemostatic gel, that is designed to stop moderate to severe bleeding within seconds.



Scan the QR code to access VETIGEL® pricing or learn more at vetigel.com



Technical

Use	VETIGEL® Hemostatic Gel is a prescription hemostatic agent for veterinary use only. VETIGEL® helps control bleeding in animals by achieving hemostasis in seconds without the need for manual pressure.
Presentation	VETIGEL® (an opaque, tan, colloidal hydrogel) is supplied in individually pouched, sterile, 5 ml syringes.
Composition	VETIGEL® is comprised of a proprietary blend of plant-based polymers: sodium alginate and poly (N-acetyl-D-glucosamine, D-glucosamine). The syringe adapter is a VETIGEL® syringe thread to Luer connector adapter, composed of autoclavable and biocompatible nylon resin or polypropylene resin. The precision tip is a 45-degree angled blunt tip dispensing needle with a Luer connector, composed of 304 Stainless Steel and Polypropylene.
Mechanism of action	VETIGEL® creates a mechanical seal at the site of bleeding and supports the formation and stabilization of a localized platelet plug.
Instructions for use	<ol style="list-style-type: none">1. Remove one VETIGEL® syringe from its pouch.2. Prime the syringe by slowly depressing the plunger until the gel reaches the end of the syringe. Using sterile gauze, wipe away any excess gel that comes out from the nozzle.3. Prepare the application site by identifying the source of bleeding and removing as much blood as possible from the site prior to application.4. Fill the intended site with the gel, starting at the deepest part and withdrawing the syringe slowly as the site fills. At the end of application, shear the gel toward the adjacent tissue in a wiping motion, to ensure the gel does not adhere to the nozzle.
Warnings and contraindications	<p>For animal use only. Not for human use. For use only by licensed veterinary surgeons. Keep out of the reach of children.</p> <p>VETIGEL® and the associated syringe adapters and precision tips are manufactured and supplied as single-use veterinary medical devices. VETIGEL® is not intended to be used as a wound-closure device. VETIGEL® should not be injected intravascularly. VETIGEL® is not intended to be used for controlling menorrhagia.</p>
Storage Conditions	Store VETIGEL® Hemostatic Gel and Dental Kit in a refrigerator (35.6-46.4F) upon receipt. All the components should be kept in the original carton to prevent damage to the packaging. Do not freeze. Gel shelf life: 24 months refrigerated.
For more information	Please contact support@vetigel.com or refer to the Instructions for Use for complete indications, contraindications, warnings, and precautions.